




# HARNESS RETAINER BRACKETS


## USER MANUAL



**Fig.1**

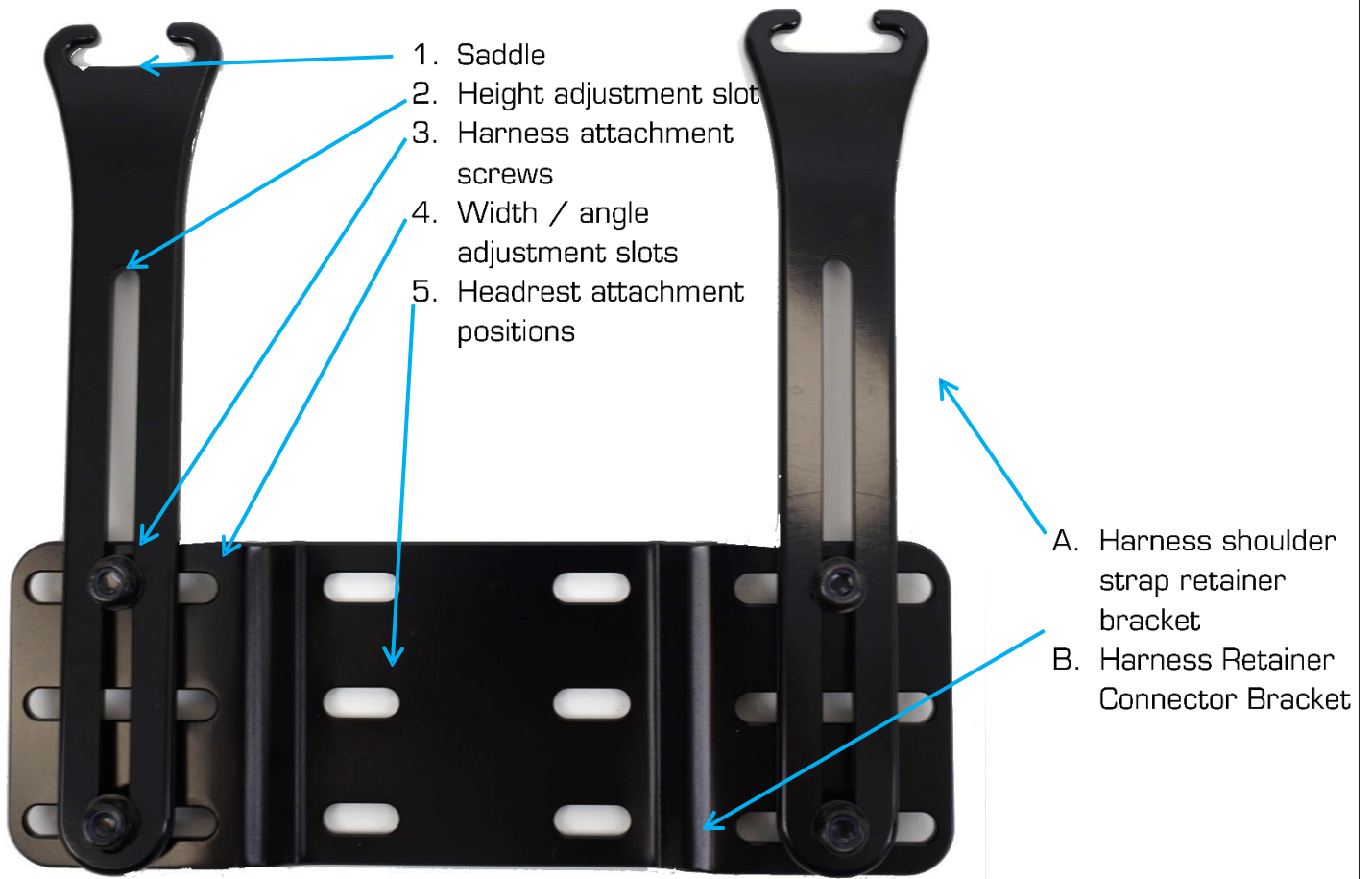

**Fig.2**

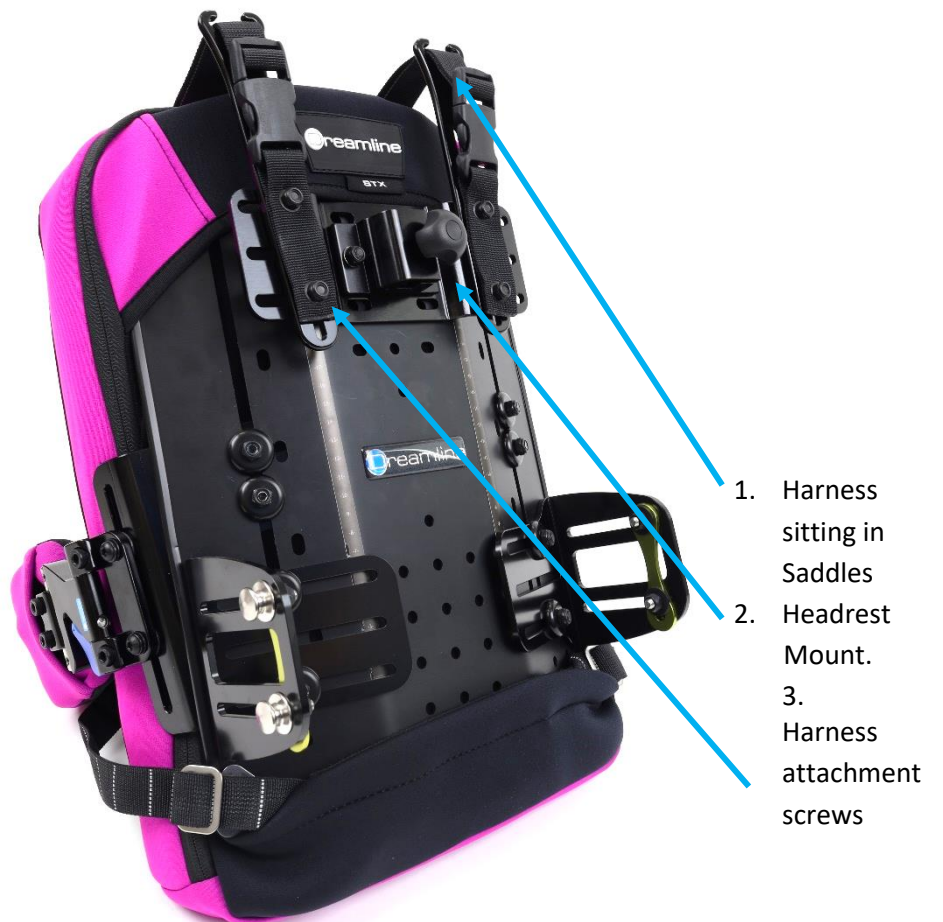

## CAUTION

- This item must only be installed or adjusted by a qualified therapist or seating technician.
- Ensure there is sufficient clearance between the shoulder straps for the occupant's neck (pressure, rubbing risk).

Harness Retainer brackets are used to hold the shoulder strap portion of a harness in position on the shoulders. They are also useful restrict any downward pull of the harness on the occupant's shoulders.

## INSTALLING SHOULDER STRAP RETAINER BRACKETS ONTO THE BACKREST SHELL

The Harness Retainer Bracket is designed to attach to both the Dreamline Support & Dreamline Contour Backrest series.

1. Attach the Harness Retainer Connector Bracket to the top / rear of the backrest shell just below the handle cut-out. Dreamline Contour & Support series backrests have pre-drilled holes for quick installation.
2. In situations where a headrest is being use, the Harness Retainer Connector bracket fits between the backrest shell & Headrest mount & is held in place by means of the headrest fasteners (Fig.2 #1).
3. The shoulder harness attaches using the Harness Attachment Screws (Fig.1 #3), (Fig.2 #2) & the shoulder straps sit in the saddles (Fig.1 #1), (Fig.2).

## ADJUSTING THE SHOULDER STRAP RETAINERS

1. The height & width of the shoulder strap retainers adjust via the harness attachment screws (Fig.1 #3).
2. Set the height of the Harness Retainer Saddles (Fig.1 #1) level with the top of the occupant's shoulders.
3. Set the width position of the Harness Retainer brackets so the harness shoulder straps are centrally positioned on each shoulder. **CAUTION:** If the Harness Retainer Brackets are positioned too close to the occupant's neck they may risk causing rubbing & pressure issues.

# Dreamline

WHEELCHAIR SEATING

by

ROLAPAL



Dr Dreamline

Check out our "How to"  
Videos Pages

Manufactured By:



**ROLAPAL Ltd**

14 Marphona Crescent, Takanini 2105 Auckland, New Zealand

Email: [info@rolapal.co.nz](mailto:info@rolapal.co.nz)

Website: [www.rolapal.co.nz](http://www.rolapal.co.nz)

EC

REP

BEO MedConsulting Berlin GmbH

Helmholtzstraße 2–9, D-10587 Berlin

