



BACK SUPPORTS

INSTRUCTIONS FOR USE MANUAL



Ignite Back Support



Contour Back Support



'Support' Back Support



S3 Back Support




STX Back Support

CONTENTS:


	Page
Items included in this package	6
Before you begin	6
Diagrams	7
Transportation warnings	10
Transportation instructions	11
Installation & occupant safety warnings	12
Intended use	12
Risks identified	12
Disclaimer	12
Product life expectancy	13
Warranty	13
Installation instructions	14
Operating the quick-release function	14
Caring for your back support	15
Cleaning instructions	15
Removing the back support cover for washing	15
Reinstalling the back support unit	16
Positioning with the STX back support	18

SYMBOLS

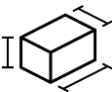	Identifies products that meet the requirements of Regulation 2017/745 / EU on medical devices		Batch code
	Indicates that a product is a medical device		Catalogue number
	Shows the manufacturer of the medical device		Date of manufacture
	WC20 & ISO7176-19 certified for forward facing use in a motor vehicle		Authorized representative in the European Community
	Product weight (kgs)		Consult instructions for use
	Max User Weight (kgs)		Warnings
	Product Dimensions		

	DREAMLINE IGNITE & PRO BACK SUPPORTS These 1-piece back supports are designed for active users. They incorporate compact and lightweight hardware and may be ordered with a carbon fibre shell. Back supports with a carbon fibre shell have not been crash tested for use as a seat in a motor vehicle.
	DREAMLINE CONTOUR BACK SUPPORT Contour back supports also incorporate a 1-piece shell design. They are for less active users.
	DREAMLINE SUPPORT, S3 & STX BACK SUPPORTS Support, S3 and STX back supports are designed for high needs users. They all utilise the same shell and hardware, only the pad differs. The Support back support has a slim profile, the S3 is a thick profile with soft foam for comfort and the STX incorporates pillows filled with Sunmate foam chips for positioning users with unconventional body shapes.

WEIGHTS & DIMENSIONS

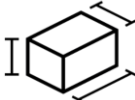ITEM			
Ignite Back Support Range	Up to 2	136kg	Up to 51cm x 51cm
Contour Back Support	Up to 2.5	136kg	Up to 51cm x 56cm
Dreamline 'Support' Back Support	Up to 3.4	136kg	Up to 51cm x 56cm
Dreamline S3 Back Support	Up to 3.4	136kg	Up to 51cm x 56cm
Dreamline STX Back Support	Up to 3.4	136kg	Up to 51cm x 56cm

IGNITE SERIES BACK SUPPORT CODES

IGNITE ACTIVE (QUICK RELEASE - ALUMINIUM)

DIBA1407	DIBA1409	DIBA1412	DIBA1414	DIBA1416	DIBA1507	DIBA1509	DIBA1512
DIBA1514	DIBA1516	DIBA1608	DIBA1610	DIBA1613	DIBA1616	DIBA1618	DIBA1708
DIBA1710	DIBA1713	DIBA1716	DIBA1718	DIBA1808	DIBA1810	DIBA1813	DIBA1816
DIBA1818	DIBA2008	DIBA2010	DIBA2013	DIBA2016	DIBA2018	DIBACUST	DIBACUSTXL

IGNITE MID (QUICK RELEASE - ALUMINIUM)

DIBM1407	DIBM1409	DIBM1412	DIBM1414	DIBM1416	DIBM1507	DIBM1509	DIBM1512
DIBM1514	DIBM1516	DIBM1608	DIBM1610	DIBM1613	DIBM1616	DIBM1618	DIBM1708
DIBM1710	DIBM1713	DIBM1716	DIBM1718	DIBM1808	DIBM1810	DIBM1813	DIBM1816
DIBM1818	DIBM2008	DIBM2010	DIBM2013	DIBM2016	DIBM2018	DIBMCUST	DIBMCUSTXL

IGNITE DEEP (QUICK RELEASE - ALUMINIUM)

DIBD1412	DIBD1414	DIBD1416	DIBD1418	DIBD1512	DIBD1514	DIBD1516	DIBD1518
DIBD1613	DIBD1616	DIBD1618	DIBD1620	DIBD1713	DIBD1716	DIBD1718	DIBD1720
DIBD1813	DIBD1816	DIBD1818	DIBD1820	DIBD2013	DIBD2016	DIBD2018	DIBD2020
DIBDCUST	DIBDCUSTXL						

IGNITE PRO ACTIVE (QUICK RELEASE - CARBON)

DPBA1407	DPBA1409	DPBA1412	DPBA1507	DPBA1509	DPBA1512	DPBA1608	DPBA1610
DPBA1613	DPBA1708	DPBA1710	DPBA1713	DPBA1808	DPBA1810	DPBA1813	DPBA2008
DPBA2010	DPBA2013	DPBACUST	DPBACUSTXL				

IGNITE PRO MID (QUICK RELEASE - CARBON)

DPBM1407	DPBM1409	DPBM1412	DPBM1507	DPBM1509	DPBM1512	DPBM1608	DPBM1610
DPBM1613	DPBM1708	DPBM1710	DPBM1713	DPBM1808	DPBM1810	DPBM1813	DPBM2008
DPBM2010	DPBM2013	DPBMCUST	DPBMCUSTXL				

IGNITE PRO DEEP (QUICK RELEASE - CARBON)

DPBD1409	DPBD1412	DPBD1509	DPBD1512	DPBD1610	DPBD1613	DPBD1710	DPBD1713
DPBD1810	DPBD1813	DPBD2010	DPBD2013	DPBDCUST	DPBDCUSTXL		

To order the fixed option add an 'F' after the letter code

DREAMLINE CONTOUR BACK SUPPORT CODES

DREAMLINE CONTOUR BACK SUPPORT (FIXED)

DCBF0810	DCBF0812	DCBF1012	DCBF1014	DCBF1214	DCBF1216	DCBF1218
DCBF1414	DCBF1416	DCBF1418	DCBF1420	DCBF1616	DCBF1618	DCBF1620
DCBF1622	DCBF1816	DCBF1818	DCBF1820	DCBF1822	DCBF2018	DCBF2020
DCBF2022	DCBFCUST	DCBFCUSTXL				

DREAMLINE CONTOUR BACK SUPPORT (QUICK-RELEASE)

DCB0810	DCB0812	DCB1012	DCB1014	DCB1214	DCB1216	DCB1218
DCB1414	DCB1416	DCB1418	DCB1420	DCB1616	DCB1618	DCB1620
DCB1622	DCB1816	DCB1818	DCB1820	DCB1822	DCB2018	DCB2020
DCB2022	DCBCUST	DCBCUSTXL				

DREAMLINE SUPPORT BACK SUPPORT SERIES CODES

DREAMLINE SUPPORT BACK SUPPORT - QUICK-RELEASE

DSB0810	DSB0812	DSB1012	DSB1014	DSB1214	DSB1216	DSB1218	DSB1414
DSB1416	DSB1418	DSB1420	DSB1616	DSB1618	DSB1620	DSB1622	DSB1816
DSB1818	DSB1820	DSB1822	DSB2018	DSB2020	DSB2022	DSBCUST	DSBCUSTXL

DREAMLINE SUPPORT BACK SUPPORT - FIXED

DSBF0810	DSBF0812	DSBF1012	DSBF1014	DSBF1214	DSBF1216	DSBF1218	DSBF1414
DSBF1416	DSBF1418	DSBF1420	DSBF1616	DSBF1618	DSBF1620	DSBF1622	DSBF1816
DSBF1818	DSBF1820	DSBF1822	DSBF2018	DSBF2020	DSBF2022	DSBFCUST	DSBFCUSTXL

DREAMLINE S3 BACK SUPPORT - QUICK-RELEASE

D3B0810	D3B0812	D3B1012	D3B1014	D3B1214	D3B1216	D3B1218	D3B1414
D3B1416	D3B1418	D3B1420	D3B1616	D3B1618	D3B1620	D3B1622	D3B1816
D3B1818	D3B1820	D3B1822	D3B2018	D3B2020	D3B2022	D3BCUST	D3BCUSTXL

DREAMLINE S3 BACK SUPPORT - FIXED

D3BF0810	D3BF0812	D3BF1012	D3BF1014	D3BF1214	D3BF1216	D3BF1218	D3BF1414
D3BF1416	D3BF1418	D3BF1420	D3BF1616	D3BF1618	D3BF1620	D3BF1622	D3BF1816
D3BF1818	D3BF1820	D3BF1822	D3BF2018	D3BF2020	D3BF2022	D3BFCUST	D3BFCUSTXL

DREAMLINE STX BACK SUPPORT - QUICK-RELEASE

DXB0810	DXB0812	DXB1012	DXB1014	DXB1214	DXB1216	DXB1218	DXB1414
DXB1416	DXB1418	DXB1420	DXB1616	DXB1618	DXB1620	DXB1622	DXB1816
DXB1818	DXB1820	DXB1822	DXB2018	DXB2020	DXB2022	DXBCUST	DXBCUSTXL

DREAMLINE STX BACK SUPPORT - FIXED

DXBF0810	DXBF0812	DXBF1012	DXBF1014	DXBF1214	DXBF1216	DXBF1218	DXBF1414
DXBF1416	DXBF1418	DXBF1420	DXBF1616	DXBF1618	DXBF1620	DXBF1622	DXBF1816
DXBF1818	DXBF1820	DXBF1822	DXBF2018	DXBF2020	DXBF2022	DXBFCUST	DXBFCUSTXL

INCLUDED IN THIS PACKAGE




Back Support x 1



Back Support Clamps (1 x pair) *

You will receive the correct sized clamps (compact or large) dependent on the size of your back support

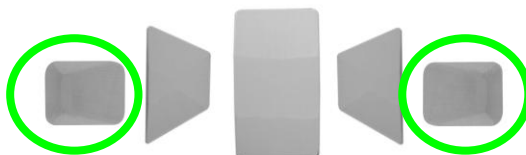

Lumbar Pad x 1

Lateral wedges x 2

Narrow Lateral Wedges x 2
(only supplied with Ignite Back Supports)

*If you purchase a fixed back support the clamps are pre-fitted to the back support.

EXTRAS



Pack of 6 x 25mm cap screws to use in the back support clamps when back support canes are less than 25mm (1") Diameter. (Only provided with large-size clamps).



Extra set of 4 Sunmate pillow Packs DCSFB46 (STX back supports only)

BEFORE YOU BEGIN - PRESETTINGS

-  Clamp release levers in their "up" (unlocked) position as shown.
-  Width adjustment nyloc nuts slightly loosened so the brackets can slide.
-  Depth adjustment nyloc nuts tightened at their furthestmost depth setting as shown (on both the left & right sides).



Fig 1



Fig 2



Fig 3



Fig 4



Fig 5

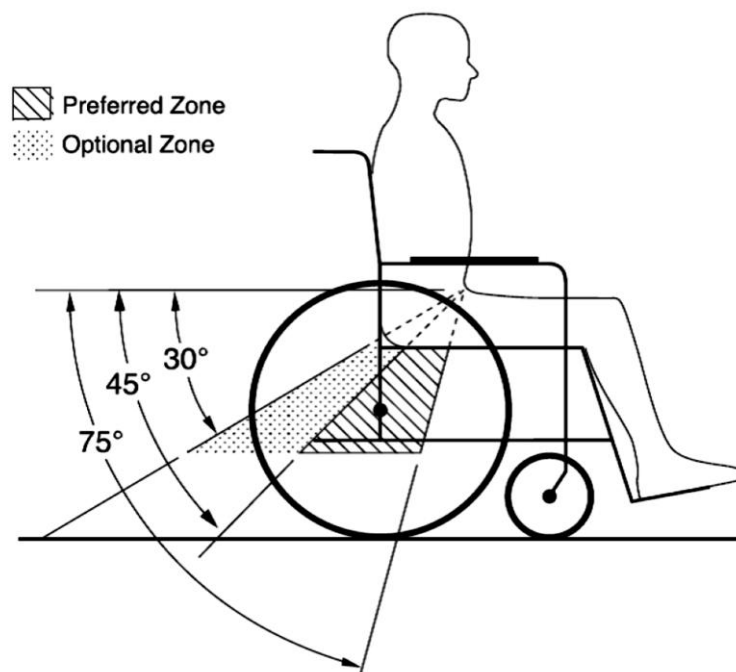


Fig 6



Fig 7


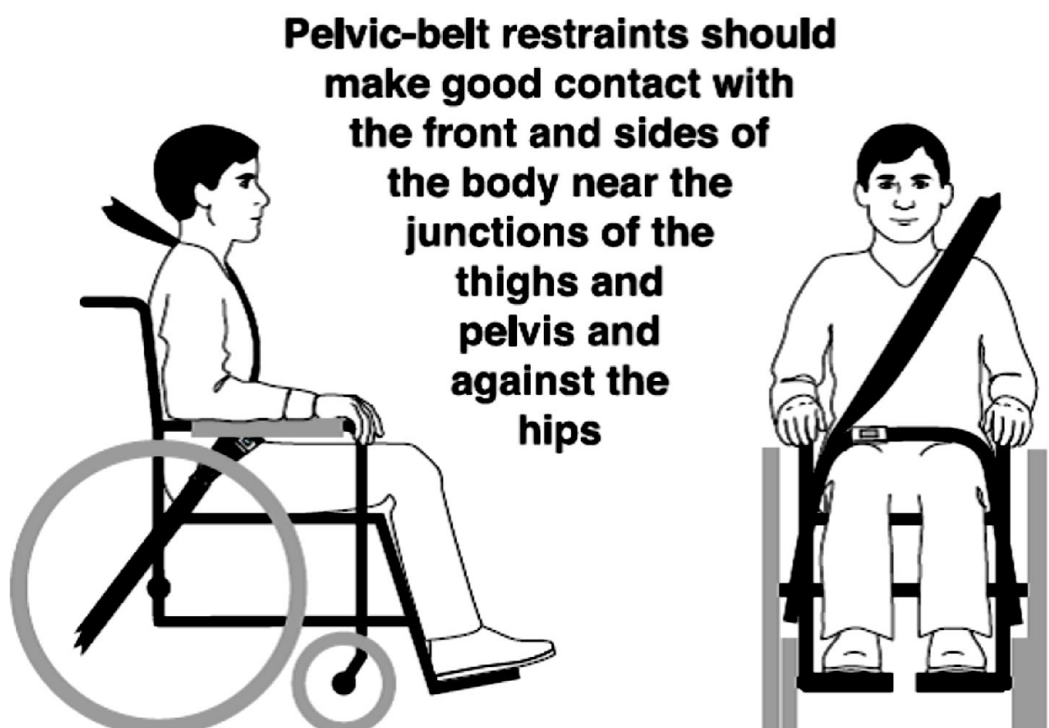



Fig 8

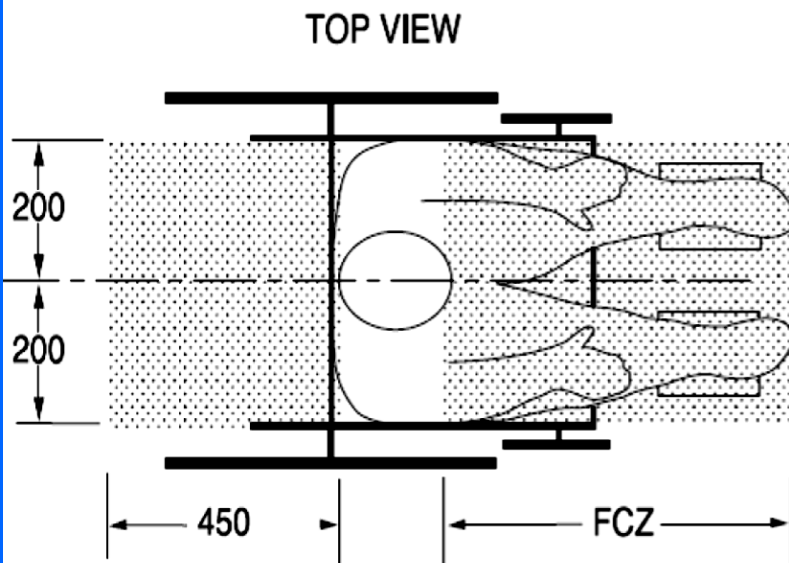

NOTES: The rear clear zone is measured from the rearmost point on an occupant's head. The front clear zone is measured from the frontmost point on an occupant's head.

Seated head height (HHT) ranges from about 1200 mm for a small adult female to about 1550 mm for a tall adult male.

Fig 9

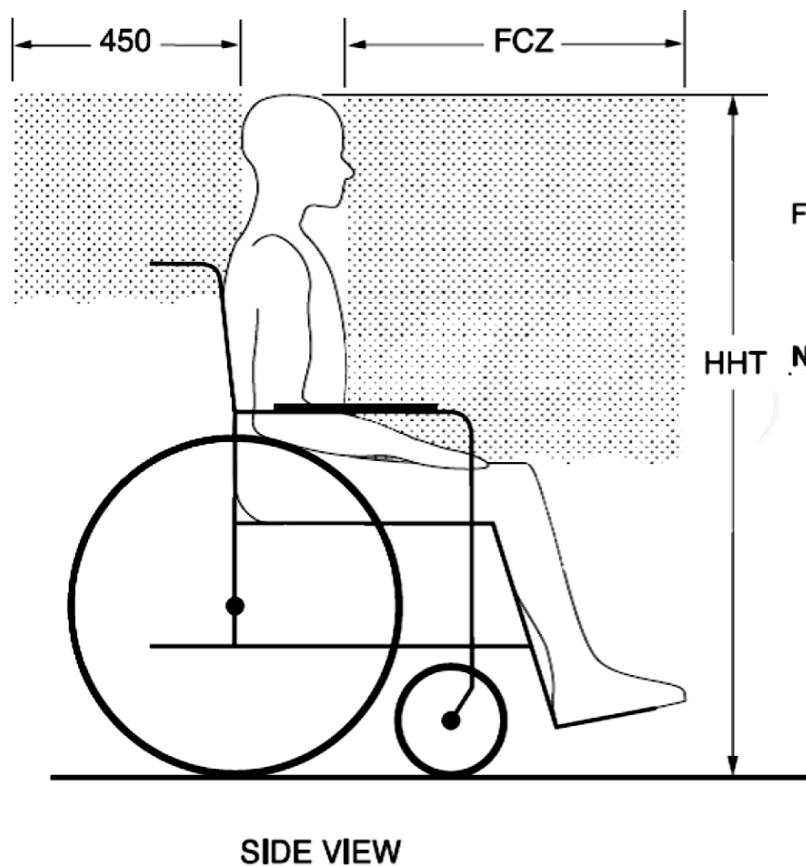

FCZ = 650 mm with upper torso restraint
= 950 mm with only pelvic restraint

NOTE: It is strongly recommended that both pelvic and upper-torso belts be used.


Recommended forward and rearward clear zones for wheelchair-seated occupants

NOTE The frontal clear zone may not be achievable for wheelchair-seated drivers.



TRANSPORTATION WARNINGS

1. When transporting occupants on their wheelchairs inside motor vehicles, Dreamline Wheelchair Seating is designed only for use with wheelchair frames that have been successfully crash tested to the frontal-impact requirements of RESNA WC-42012, Section 19.
2. When used as a seat for the occupant inside a motor vehicle, Dreamline Wheelchair Seating must only be used on wheelchair frames that provide four securement points and pelvic belt anchor points that conform to the requirements of RESNA WC-4:2012, Section 19.
3. When the Dreamline seating system is used for occupancy inside a motor vehicle, it is designed specifically for wheelchairs that are forward facing inside the motor vehicle.
4. When the Dreamline Seating system is used for occupancy inside a motor vehicle it must only be used in accordance with the wheelchair manufacturer's instructions.
5. Back supports with adjustable recline angles should not be tilted back more than 30 degrees to the vertical during travel in vehicles unless necessary for the medical & postural needs of the occupant.
6. If the back support must be adjusted to an angle that is greater than 30 degrees to the vertical during travel in a motor vehicle, the upper shoulder belt anchor point should be adjusted rearward to maintain belt contact with the wheelchair occupant's shoulder and chest.
7. Wheelchair mounted rigid trays that are not specifically designed for use during travel in motor vehicles should be...
 - a. Removed and secured separately in the vehicle, or
 - b. Secured to the wheelchair so they will not break free in a crash and
 - i. Be positioned with a gap of at least 75mm (3") between the back edge of the tray and the wheelchair occupant's abdomen and/or chest so as not to interfere with proper belt restraint use, and
 - ii. Have energy absorbing padding placed between the back edge of the tray and the wheelchair occupant.
8. The use of a postural pelvic belt attached to the wheelchair base or seat frame is encouraged during travel, but these belts should be positioned so they don't interfere with the proper positioning of crashworthy belt restraints, and they should not be relied on for occupant protection in crash situations unless the postural belt has been designed to comply with, and perform to, the requirements of 4.6, 5.2, and 5.3 of RESNA WC-4:2012, Section 19 (i.e. with WC19).
9. People who use wheelchairs should transfer to the vehicle seat and use the vehicle installed occupant-restraint systems when this is feasible.
10. There must be sufficient clear space around the wheelchair for the wheelchair occupant and other occupants inside the vehicle to remain safe during travel. Recommendations are made in Fig 8, 9 for Head Height (HHT) and rear space of 450mm from the users back with a Frontal Clear Zone (FCZ) of 650mm with pelvic / upper torso restraint or 950mm if there is only a pelvic restraint. This may not be achievable for wheelchair seated drivers. Wheelchair seated drivers must comply with the modified vehicle and transport approved restraint manufacturer's instructions.

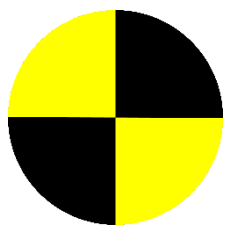

TRANSPORTATION INSTRUCTIONS

The following components have all been tested & are both WC20 & ISO7176-19 certified for forward facing transportation with an occupant in a motor vehicle; Quick-release Seat Pan, All Dreamline Cushions, Dreamline Contour, Support, S3 & STX Back supports, Quick-release swing-down Hip guides & Arm supports, Swing-down Pommel, both fixed & Swing-away laterals & Dreamline Axis Head supports.

Dreamline Wheelchair Seating fully complies with RESNA WC-4:202, Section 20 & when used with a wheelchair frame which complies with RESNA WC-4:2012, Section 19 if used as a transportation device for an occupant in a motor vehicle.

Dreamline Wheelchair Seating has been designed to accommodate the proper use of vehicle anchored belt restraints with a rating of 16 out of 16 & pass level of A+ (Excellent): a) ease of properly positioning vehicle anchored belt restraints on the wheelchair passenger & b) the degree to which proper positioning of the belt restraints can be achieved.

1. The pelvic belt transport approved restraint should be worn low across the front of the pelvis near the junctions of the pelvis and thighs.
2. The angle of the transport approved pelvic belt restraint should be between 30 degrees to 75 degrees relative to the horizontal and ideally between 45 degrees to 75 degrees to the horizontal as shown in Fig 5.
3. The transport approved belt restraint buckle of three-point belt restraints should be placed in contact with the occupant's body and away from wheelchair components & should not be routed outside of the wheelchair wheels or over the wheelchair arm supports (Fig 6,7).
4. The junction of the shoulder belt & the pelvic belt of the transport approved three-point belt restraint should be located near the hip opposite the shoulder over which the diagonal belt crosses and not near the midline of the occupant (Fig 7).
5. Upper torso transport approved belt restraints should fit directly over, and in contact with, the middle of the shoulder (Fig 7)
6. Transport approved belt restraints should be adjusted to fit as snugly as possible, consistent with user comfort.
7. Transport approved belt restraints should not be worn twisted in a manner that reduces the area of contact of the belt webbing with the occupant.

ISO 16840-4

Dreamline Wheelchair Seating has been tested to comply with the requirements of ISO16840-4. Dreamline Wheelchair Seating will only comply if the corresponding wheelchair frame complies with the requirements of ISO7176-19. The frame must include anchorage points which comply with the requirements of ISO7176-19. Dreamline Wheelchair Seating have been designed and tested for forward facing travel in a motor vehicle. Dreamline Wheelchair seating will only comply if the corresponding wheelchair tie-down & occupant restraint system (WTORS) complies with ISO 10542-1.



INSTALLATION & OCCUPANT SAFETY

- This Dreamline Back Support must only be installed and configured by a qualified seating therapist or technician.
- Fully read the instructions before installing the Back Support.
- The Restrictor Plate (Fig 1) must never be removed or modified to ensure the safety of the user.
- Do not use this Back support if it has become loose, broken, not in the desired position, or not working properly.
- Always ensure the Back Support is locked in place before using.
- Do not use if redness or pain develops in the occupants back.
- Do not modify Rolapal products as this may affect the strength of the item making it a potential safety risk to the occupant.

INTENDED USE

This item is intended for persons with disabilities as a Back support for a wheelchair seat. If there are any questions about product installation or compatibility with the wheelchairs, ROLAPAL customer service should be consulted.

Indication: Partially or permanently non-ambulatory limited to a sitting position

Contra-indication: None

CAUTION - RISKS IDENTIFIED

- Take care when removing this item from the package and general handling risk of entrapment / impingement.
- This back support can be positioned in many ways, take care to ensure the desired outcome is achieved for the occupant.
- Keep away from heaters and open flame – risk of fire.
- The Back support pad cover is removable for cleaning. Do not use if these are not correctly in place behind the user.
- Take care when using this back support in direct sunlight. Parts may heat up and increase risk of burns to the occupant.
- Metal or dark coloured may become very hot if left in direct sunlight. Burn risk.

REPORTING OF INCIDENTS

All serious incidents occurring in relation to the product must be reported to ROLAPAL Ltd and your competent authority!

DISCLAIMER

Because of the infinite adjustability of the Back support, Rolapal accepts no responsibility for the therapeutic outcome for the occupant. This is the responsibility of the prescribing seating therapist or technician.

REUSABILITY

The product is basically suitable for reuse. The reuse handover takes place after the product has been checked by the specialist trade.

SPECIAL STORAGE/HANDLING CONDITIONS

Do not store in environments above 50°. Do not store close to roofs to avoid heat. If not in use, please store in dust free environments.

PRODUCT LIFE EXPECTANCY

This item is expected to last between 2 – 5 years under normal use but this can change dramatically between users depending on the length of time in use, environment & weight of the user.

END OF LIFE RECYCLING

At the end of its useful life, all metal components are easily removed for recycling. Foam inner and cover are general waste disposal.

GUARANTEE

Dreamline Back Supports are manufactured to a very high standard and carry a guarantee against defects in materials and craftsmanship for a period of 18 months.

Any item found to be defective will be repaired or replaced at the discretion of Rolapal Ltd.

Goods must be returned to Rolapal Ltd for assessment and the new repaired or replaced item will be sent free of charge.

This guarantee does not cover items which are damaged from general “wear and tear”, misuse or negligence.

MAINTENANCE

Perform weekly checks on the back support & wheelchair frame to ensure the back support attaches correctly to the wheelchair frame, no parts are loose, broken or missing. If any component appears to be faulty, stop using the back support immediately & contact your seating therapist or technician. Keep mechanisms clean and free from dust.

BACK SUPPORT INSTALLATION INSTRUCTIONS

Attaching the back support clamps to the wheelchair canes:

1. Temporarily hold the back support in position to over the wheelchair frame to determine the preferred height to place the back support clamps on the wheelchair frame. With heavy occupants, for a robust attachment to the wheelchair frame and general stability of the back support, the best height for the 2-point attachment method attachment is approximately mid back support height (but also take into consideration lateral position etc). If the occupant is deemed too heavy for a 2-point attachment system, use 4-point attachment instead (as in Fig 4).
2. Back support clamps may be attached to the front or rear of the wheelchair frame, but the clamp saddles must always be stationed on the inside of the wheelchair frame with the open portion of the saddle always facing upwards with the blue lock tab facing rearwards (Fig 1). The saddle attaches to the back support clamp by means 2 screws on the inside of the clamp which can be removed with a 4mm Allen Key to change their orientation in view of achieving the different fore or aft set-up modes (for this, the saddles will need to be removed from the clamps and rotated). The left & right saddles must be parallel with each other and attached to the wheelchair frame at the same heights (Fig 1).
3. Once the clamps are positioned as desired on the wheelchair frame & they are parallel with each other & at the same height, fully tighten all 3 cap screws on each clamp using a 5mm Allen Key to firmly secure it to the wheelchair frame. Tighten the screws over & over until fully tight. If the back support cane is less than 25mm diameter, the cap screws will have to be exchanged with the shorter cap screws supplied.

Attaching the Back support to the clamps:

1. Slide the two L-brackets on the back support (circled in blue - Fig 2) so the pins (circled in green – fig 2) insert into the saddles on the clamps then press the blue lock lever on each saddle downwards to lock in position.
2. Move the back support shell laterally and up / down to centralise it between the clamps and position it at the correct height as desired then tighten the four nyloc nuts (circled in blue – Fig 2) using a 10mm spanner to lock the height & width settings of the back support.

Setting the Back support depth & Angle:

1. The back support depth and angle are adjusted by using a 10mm spanner to loosen the 2 nyloc nuts which holds the back support pins in position on each L-bracket (circled in blue – Fig 3).
2. Once the screws are loosened, move the back support forward / backwards and set the tilt angle then tighten the two Nyloc nuts on each L-bracket to secure in place again.

Operating the back support quick-release mechanism:

To fit the back support onto the wheelchair frame, insert the two pins (circled in green – Fig 2) into the saddles on the back support clamps (Fig 1) then press the blue lock lever downwards until it “clicks” to lock.

To release, lift the blue lock lever upwards until it “clicks” in position and lift the back support off.

CARE INSTRUCTIONS

Removing the back support pad from the back support shell

To separate the back support foam unit from the shell, lift the flaps on the cover at the rear of the back support (with the ignite & Contour Back supports lift the flaps right over to the front surface of the back support), then lift the loop tabs at the top & bottom and pull back support pad away from shell.

Removing the cover from the pad

All back support outer covers are removable for washing. Unzip cover and remove the inner.

Cleaning the Back support

- Outer Cover - Machine washable to 50°C. Drip dry only. Do not Iron. Do not use bleach or other strong cleaners.
- Water Resistant Cover - Wipe down only with soapy water and drip dry thoroughly before re-installing the outer cover.
- STX back support Inner Cover – remove Sunmate liner from front pocket then wash the cover normally.
- Foam Inner - Do not wash the foam inner. If it accidentally becomes wet, separate it from the cover & drip dry it thoroughly before re-installing the cover.

Reinstalling the back support pad to the back support shell

ORIENTATION – Support, S3 & STX Back Supports

The top of the foam has a gently curved profile. The zip runs around the back edge over along the sides & over the top edge of the back support. The top cover flap has 3D Spacer fabric inserts to distinguish it from the bottom. The stretchy fabric on the cover is the face surface of the back support. the following foam surfaces must face the front; blue viscoelastic foam (Support Back support), yellow profile cut foam (S3 Back supports), pillow packs (STX Back support). With the STX Back support, the inner cover has a foam insert inside the front surface of the cover.

ORIENTATION – Contour Back Supports

The blue foam surface of the foam faces the front. The stretchy fabric is the front surface of the cover. The narrow portion of the back support foam is the top. The cover has a hidden zip which runs along the bottom edge of the back support.

ORIENTATION – Ignite & Ignite Pro Back Supports

Ignite back supports are reversible. One edge is more curved than the other. To get the orientation correct, match the Labels on the foam inner at the same point as the labels on the cover & on the shell (the labels are always put on the more rounded or narrower end).

Fold this forward over the front of the back support pad before installation begins. With the Contour Back support the narrow portion is the top.

Attachment – Support, S3 & STX Back Supports

1. With the cover unzipped, attach the rear panel of the cover to the hook surface on the rear side of the foam inner. Once this placed well, zip up the covers. Under the top flap at the rear of the cover there is a loop tap which attaches to the hook surface at the top of the back support shell – attach this first,
2. Slip the top flap of the cover over the top edge of the shell & press into place against the front surface of the shell.
3. Lift the bottom flap over the bottom edge of the back support shell & attach the loop inside of the flap to the hook on the back support shell.

Attachment – Contour Back Supports

1. Fold the foam & insert it with the narrow portion first, from the bottom of the cover pushing it up & manipulating it until it fits to all the edges of the cover, then zip up the cover.
2. Slip the top flap of the cover over the top edge of the shell & press into place against the front surface of the shell.
3. Lift the bottom flap over the bottom edge of the back support shell & attach the loop inside of the flap to the hook on the back support shell.

Attachment - Ignite & Ignite Pro Back Supports

Fitting the cover to the foam

1. Flip the cover flaps forward to the front of the cover.
2. Sit the cover face down so the zip is exposed. The blue side of the foam is the front face. Sit the foam face down beside the cover.
3. Unzip the cover & flip the zip panel away to access inside of cover.
4. Fit the foam inner inside the cover beginning at the rounded (labels) end first, then fitting in the laterals then finally the fitting the remaining end. Adjust the cover so the laterals are fitting nicely. Centre marks at each end of the foam line up with centre stitching & labels on the cover. Pull the zip panel up until the perimeter seam sits at the rear edge of the foam inner then press the zip panel down centrally onto the reciprocal hook panel on rear of the foam. Zip up the cover & make any final adjustments.

Sequence of fitting the pad to the shell.

1. Flip the cover flaps forward to the front of the pad
2. Sit the pad face down on a flat surface & the shell face down on top of it.
3. Fit – Attach the Loop Tab from under the cover flap of the rounded (labels) end of the pad onto the reciprocal hook positioned midline at the rear of the rounded end of the shell then pull the opposite loop tab up to the reciprocal hook at the other end of the shell and attach. This ensures the pad is centrally positioned onto the shell.
4. Fold the laterals on the pad inwards away from the shell & reattach them in position onto the shell.
5. Flip the cover flaps back over into position covering the shell beginning from one lateral working across the back to the other lateral.



- Zip Panel on cover.
- Hook receptive Loop central panel on inside of cover.
- Hook strip on rear side of foam inner.
- Ignite Logo to be centred over Hook strip on foam inner.
- Rear flaps (top / bottom).
- Perimeter seam



POSITIONING WITH THE STX BACK SUPPORT

The STX Back support offers comfortable positioning for high needs occupants who may have an unconventional body shape.

The STX Back support pad incorporates an inner cover & inside the inner cover there is a foam pad which has multiple pillow packs attached.

The inner foam pad may be removed from the covers to make it easier to adjust the shape with the pillow packs.

The pillow packs (order code: DCSFB46) are attached to the inner pad with hook & Loop fasteners & may be repositioned, removed, or layered up to best match the body shape of the occupant. The pillow packs can be set at any orientation or bunched up before being attached for the best fit. Four extra pillow packs are included with each back support.

To limit access to the inner, inhibiting unwanted adjustment of the pillow packs position, the installer may choose to cable-tie the dual zip sliders of the inner cover together.





WHEELCHAIR SEATING

Posture adaptive

Pressure reactive

Performance effective



Ignite

SPLASH

Dreamline

WHEELCHAIR SEATING



Woven Air

snooza



Quality Assurance

Rolapal Ltd is an ISO 9001, 14001 & 45001 certified company!



WHEELCHAIR SEATING

by

ROLAPAL



Dr **D**reamline

Check out our “How to”
Videos Pages

Manufactured By:




ROLAPAL Ltd

14 Marphona Crescent, Takanini 2105 Auckland, New Zealand

Email: info@rolapal.co.nz

Website: www.rolapal.co.nz

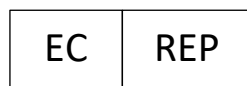

BEO MedConsulting Berlin GmbH

Helmholtzstraße 2–9, D-10587 Berlin

